



123 Lincoln Boulevard, Middlesex, NJ 08846
732-356-1493
service@phillips-safety.com

Instructions for Care- Leaded Shields for Portashields

SKUs: LB-MB (multiple sizes)

Cleaning:

- Clean with a damp cloth and mild detergent. If you need to scrub the shield, use a soft toothbrush
 - Avoid immersing the shield in water or putting it under a faucet.
 - Avoid harsh cleaning agents like bleach, IPA/ethanol .
 - Do not machine wash.

Disinfecting:

- The following disinfecting agents are acceptable for application with a damp cloth, then allow to dry:
 - Hydrogen peroxide, 6% solution
 - Water based QAT's – or quaternary ammonium cleaners/disinfectants – are acceptable provided they are substantially alcohol-free and made up in dilute water-based forms. Follow manufacturer's instructions for make-up/ dilution.
- Do not autoclave.

Storage:

- Shields should be stored flat when not on base.
- Do not fold! Folding can lead to cracks in the protective material.